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**📄 Blood glucose control solution**

**Package contents**

- 1 vial with 4 mL control solution MEDIUM for **Beurer GL30, GL32, GL34 and BGL40, BGL60 blood glucose monitors**
- 1 vial with 4 mL control solution HIGH for **Beurer GL30, GL32, GL34 and BGL40, BGL60 blood glucose monitors**
- This package insert

**⚠️ WARNING**

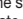
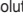
- Not for internal use. The control solution is an in-vitro diagnostic agent. This means that it is intended for function tests outside the body only.
- The results range printed on the test strip container is only applicable to the control solution. **It is not a recommended value for your blood glucose level.**
- If you receive repeated results outside the specified range with the control solution, **do not continue using the system to measure your blood glucose level.** Contact our Customer Service department.

**Proper use**

The control solution (MEDIUM, HIGH) is used in conjunction with **Beurer test strips** (REF 457.00, 464.00) and the **Beurer GL30, GL32, GL34 or BGL40, BGL60 blood glucose monitors**. It is used to check that the monitor and test strips operate together correctly and whether the test is conducted correctly. The test with the control solution (MEDIUM, HIGH) must be conducted regularly to ensure that your test results are accurate, and always when you open a new container of test strips. Every vial contains enough solution for about 75 tests. The control solution (MEDIUM, HIGH) is suitable for self-testing.

**📖 Instructions for Use and Storage**

Read carefully the following information on the use and storage of your control solution. The function test with the control solution will only be reliable if you follow all instructions carefully.

- Never use control solutions from other manufacturers. It will be impossible to check that your Beurer blood glucose monitor operates correctly with other control solutions.
- Do not use control solution after the use-by date. If expired control solution is used, function test results may be inaccurate. The use-by date can be found on the vial beside the hourglass icon .
- The control solution can be stored for three months after opening the vial. Write this use-by date (date of opening + 3 months ) on the label. The storage time is reduced if it extends beyond the use-by date (see date beside the hourglass icon .
- Do not continue to use the control solution if one of the use-by dates (/) has expired.
- Monitor, test strips, and control solution must be brought to room temperature (20 °C to 25 °C) before the test.
- Store the control solution tightly closed at room temperature below 30 °C. Do not store under refrigeration. (2 °C – 30 °C)
- Store the control solution out of reach of small children.
- Do not store the control solution in the medicine cabinet. It may be confused with medications for internal use.

**How it works**

The control solution contains a specific proportion of glucose, which reacts with the test strip. A test with control solution is similar to a blood test. The only difference is that a drop of control solution is used instead of a drop of blood. The test result with the control solution must be within the measurement range. The result range is printed on every test strip container.

**When is a test with control solution recommended?**

- Once a week.
- After opening a new test strip container.
- If you think the monitoring system or the test strips may be faulty.
- If you feel that the blood glucose monitor readings do not reflect how you feel.
- If you have dropped the monitor or it has been subjected to any mechanical stress.
- In the event of repeated unexpected blood glucose measuring results.
- If the test strips have been exposed to temperatures outside the specified storage conditions (2 °C – 30 °C).
- At any time if you suspect that the glucose test results are incorrect.

One single test with control solution with a result within the results range is sufficient.

- The control solution may stain your clothes. In the event of staining, wash clothing with soap and water.
- Seal the control solution carefully after each use.
- Any excess control fluid dispensed must not be drawn back into the container.

**Checking the monitor with control solution**

Read the Instructions for Use for your monitor to conduct the function test.

**Expected results**

At room temperature the test results with the control solution should be within the range printed on the test strip container for approximately 95 % of all tests. See the Instructions for Use for your monitor for the causes of out-of-range test results and how you should respond.

Note that conclusions about changes in accuracy cannot be drawn from where the test result stands in the range. As long as a test result is within the expected range, the blood glucose measurement conforms to the accuracy given in the specifications.

**Chemical composition**

The control solution is a red solution with the following D-glucose level (in percentage shares):

Ingredients	Control solution MEDIUM	Control solution HIGH
D-glucose	0.08 %	0.19 %
Non-reactive constituents	99.92 %	99.81 %

**Disposal**

**⚠️ WARNING**  
Strictly observe the applicable regulations when disposing of test strips and control solution.

**Controls**

This product conforms to the requirements of the European IVD Directive (98/79/EC).

**Customer service address**

Please contact our Customer Service department if you have any questions. Please find the service address in the attached address insert.

OUR OBLIGATIONS TO YOU: Our mission is to supply you with high-quality health products and the best customer service. If you are not completely satisfied with this product, please contact our Customer Service department.

**Where do you get control solution?**

You can get it from your pharmacy without a prescription or wherever the Beurer blood glucose monitor is sold. If you have any more questions on getting control solution please contact Customer Service.

4 ml control solution MEDIUM,  
4 ml control solution HIGH  
REF 457.09

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Beurer GmbH,